

SHEAR COMFORT ARM SUPPORT PROTECTORS



SKU: SHE-6648

SPECIFICATIONS

Material: XD1900 Fabric-Backed Medical Sheepskin

All Shear Comfort sheepskins have the ability to be washed at 80° C, effectively killing bacteria without shrinkage. Our sheepskins are warranted against being washed 50 times at this temperature without any detrimental effects to the item. For best use we recommend using Shear Comfort Wool Care Shampoo with conditioner.

Drying instructions

Shear Comfort products may be tumble-dried warm, without exceeding 60° C. To air-dry Shear Comfort overlays, hang fleece-side down and away from direct sunlight or heat sources.

To air-dry Shear Comfort footwear or targeted protectors, release all fastenings and place fleece-side down on a drying rack and away from direct sunlight or heat sources.

Do not iron

Do not bleach

Specific Washing instructions



As our Shear Comfort Original items are backed with natural sheepskin, they require a slightly different washing technique from the XD1900, which is backed with fabric.

Please follow the washing instructions for your chosen item.

Shear Comfort Natural Sheepskin

It is recommended that Shear Comfort natural sheepskin items are washed with the Shear Comfort Wool Care Shampoo with conditioner, which contains antibacterial tea tree oil and is effective against dust mites. If an alternative is used, it must be a non-ionic liquid detergent which has no enzymes, phosphates, peroxides, alkali or bleach.

Domestic washing instructions

On a gentle cycle, wash your Shear Comfort product with approximately 20ml of Shear Comfort shampoo or approved detergent. Water must not exceed 80°C.

Industrial Washing instructions

Wash on a gentle cycle wash for approximately 10 minutes in hot water up to 60°C using Shear Comfort shampoo or approved detergent.

Rinse for approximately 3 minutes in warm water at approximately 35°C

Wash on gentle cycle without detergent for approximately 8 minutes in water up to 80°C for thermal disinfection.

Rinse for approximately 3 minutes in cool water.

Shear Comfort XD1900

Shear Comfort XD1900 does not require special shampoo, and can be washed with normal household detergent up to a temperature of 80°C, although you can use Shear Comfort Wool Care Shampoo to condition the wool and utilise the fungicidal and matricidal properties of the shampoo.

XD1900 products can be laundered in the same way as Shear Comfort Natural sheepskins.

WARRANTY

One year warranty from date of purchase. T&Cs apply

PRODUCT DESCRIPTION

The Shear Comfort Arm Support Protectors have been designed to provide comfort for uncomfortable arm supports on chairs and wheelchairs. They can also be used to prevent arms from slipping off chairs and wheelchairs, as sheep's wool yields to accommodate the arm, gently holding it in place.

For wheelchair users, the Shear Comfort Arm Support Protectors provide added support, especially for those who use the arm supports to shift their body weight.

By using the adjustable Velcro straps, the Arm Support Protectors can be attached to any arm support quickly and easily, enhancing comfort on your plain plastic, vinyl or leather chair.

Made with natural properties of fabric-backed medical sheepskin wool, unique to Shear Comfort, ensures that comfort and protection is provided. The moisture dissipation qualities of the wool ensure any perspiration is effectively wicked away from the skin.

All Shear Comfort products are machine washable and are guaranteed against deterioration for up to 50 washes.

Please note: As these are hand made sometimes there is a small dispatch delay of 1-2 weeks as they need to be made up.